

Transparent Dressing

Polyurethane Transparent Dressing

Size: 6cm x 7cm, 10cm X 12cm



■ STRUCTURE



Easy to use & change

Lable Paper
— Record Information

**Highly Breathable
PU Film**

- waterproof and breathable
- soft and anti-bacteria

**Acrylic Pressure-sensitive
Adhesive**

- medical grade adhesive
- hypoallergenic



Transparent Dressing

Polyurethane Transparent Dressing

Size: 6cm x 7cm, 10cm X 12cm

Transparent Dressing are commonly used in hospitals, clinics, and other healthcare settings to ensure the safety, comfort, and effectiveness of IV therapy. Healthcare providers should follow best practices for securement and dressing application to maximize the benefits of these devices and minimize complications related to IV catheter use



Advantages


- Securement
- Protection
- Visibility
- Comfort
- Stabilization
- Minimize Tension
- Ease of Application
- Reduced Discomfort
- Facilitate Site Inspection
- Patient Education

How to Apply

Prepare the Work Area

- Ensure that you're working in a clean and well-lit area.
- If the IV insertion site appears dirty or if there are any signs of contamination, clean the area with sterile saline or an antiseptic wipe
- Gently peel the protective backing from the transparent IV fixator, exposing the adhesive side.
- Add a little bit of body textUsing the other hand, carefully apply the adhesive side of the IV fixator over the catheter hub and the insertion site. Ensure that the catheter remains securely in the vein.
- Press down firmly along the edges of the fixator to adhere it to the skin. Make sure there are no wrinkles or air bubbles under the fixator.

Usage Steps

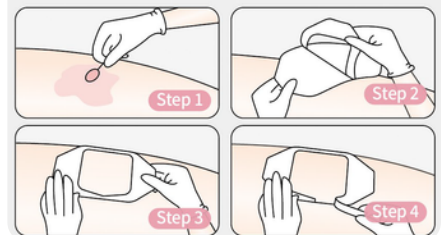

Wear time: Up to 3 days